

# DIGITAL IMPRESSION CONNECTION GUIDE

**3shape**   
TRIOS Ready

## 3 SHAPE TRIOS

1. Find "Add connections"
2. Add our 3shape email [jerry@vitalitydentalarts.com](mailto:jerry@vitalitydentalarts.com) and send the request.
3. We will accept the request within 24 hours.

**iTero**<sup>®</sup>

## ITERO

1. Find "Add preferred lab"
2. If Vitality Dental Arts is unavailable, please call iTero directly and have them add Vitality Dental Arts (ID#10093).

**sirona**  
CERTIFIED LAB

## SIRONA CEREC

1. Find "Add favorite lab"
2. Add Vitality Dental Arts (Zip Code 60123) from the drop down menu.

**Carestream**  
DENTAL

## CARESTREAM

1. Find "Add connections"
2. Add our Carestream email [anna@vitalitydentalarts.com](mailto:anna@vitalitydentalarts.com) and send the request.
3. We will accept the request within 24 hours.

**PLANMECA**  
E4D TECHNOLOGIES

## PLANMECA E4D

1. Send all scans through "DDX" (Henry Schein Portal)
2. Emailing or uploading the STL files to us directly is also an option.

1-800-399-0705  
[www.vitalitydentalarts.com](http://www.vitalitydentalarts.com)